

SARS-CoV-2 Antigen Control

INTERPRETATION OF TEST RESULTS

SARS-CoV-2 Antigen Control : Positive		
Result	Interpretation	Follow up
Pass	Positive result for Ag	
Fail	Negative result for Ag	Retest
Invalid	No Control (C) Line	Retest

SARS-CoV-2 Antigen Control : Negative		
Result	Interpretation	Follow up
Pass	Negative result for Ag	
Fail	Positive result for Ag	Retest
Invalid	No Control (C) Line	Retest

Note: Use new test devices and new control for retest.

LIMITATIONS

1. This product is provided for quality assurance purposes only.
2. Incorrect storage conditions or use of expired reagents may produce erroneous results.
3. Do not use this product following the expiry date.
4. Alterations in physical appearance may indicate instability or deterioration of this product. Discard this product if there is evidence of microbial contamination.



SD Biosensor, Inc.

Head office : C-4th&5th, 16, Deogyong-daero 1556beon-gil, Yeongtong-gu, Suwon-si, Gyeonggi-do, 16690, REPUBLIC OF KOREA

Manufacturing site : 74, Osongsaengmyeong 4-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, 28161, REPUBLIC OF KOREA

Distribution by:

Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Roche order number:

09338322190



Authorized Representative

MT Promed Consulting GmbH

Altenhofstrasse 80, 66386 St. Ingbert Germany

CANADIAN IFU

L24COVC92EN01R0

Issue date: 2021.02



Reference number



In vitro Diagnostics



Consult Instructions for Use



Contains Sufficient for \leq Tests



To indicate the temperature limitations in which the transport package has to be kept and handled.



Do not re-use



Use by



Batch code



Manufacturer



Date of manufacture



Authorised representative in the European community