

Umonium 38 Instructions for use

Product: UMONIUM 38 DECON SOLUTION (07007833001)

Indications: Cleaning and disinfection of medical devices before sterilization. Cleaning and disinfection of non-invasive medical devices and surfaces of non-invasive medical devices.

Instructions for use:

Cleaning / Pre-disinfection: Use at 0.5% (25mL / 5mL of water). Immerse then brush / use ultrasound or wet wipe to remove any dirt. Rinse thoroughly with water or wipe surfaces with a water-dampened swab.

Disinfection: Use at 2.5% (25mL / L of water) on clean devices. Apply by wet wiping or soaking, allowing a minimum contact time of 10 minutes. For invasive medical devices: Rinse with water. For non-invasive medical devices: Let dry or rinse with water (or using a wipe moistened with water) if necessary.

Precautions for use: Product an irritant to the eyes and skin. Avoid contact with the skin. In case of contact with the eyes, wash immediately with plenty of water and consult a specialist. Avoid the release of the pure product into the environment. Consult special instructions and medical safety data sheet.